OPUS™ NASAL PILLOWS MASK PATIENT INSTRUCTIONS FOR USE

THE OPUS™ NASAL PILLOWS MASK IS INTENDED FOR MULTIPLE PATIENT OR SINGLE PATIENT ADULT USE BY INDIVIDUALS WHO HAVE BEEN DIAGNOSED BY A PHYSICIAN AS REQUIRING CPAP OR BI-LEVEL VENTILATOR TREATMENT IN THE HOME, HOSPITAL, OR OTHER CLINICAL SETTING.

FITTING YOUR MASK

- Remove Plastic Cover from Silicone Pillows.
- Hold the Mask Base (A) and Silicone Pillows (C) in one hand, with the rigid arms of the Headgear (D) pointing towards your ears.
- Gently press the Silicone Pillows (F) up against the bottom of your nose, making sure the holes align with your nostrils.
- Hold the Mask Base (A) with one hand and pass the Headgear (D) straps over your head. The rigid part of the Headgear should finish just above your ears and one strap should sit at the back of your head and the other on the top of your head.
- Adjust the strap tension at the back and then at the top so that the Headgear rests comfortably on your cheek and the Silicone Pillows fit securely into your nostrils.
- Connect the mask to the CPAP/Bi-Level machine set at the prescribed pressure.

FITTING TIPS

- If leaks occur around your nostrils, try repositioning the Silicone Pillows (C) and adjusting the tension on the two Headgear straps.
- If the Silicone Pillows (C) are resting heavily on your upper lip, try changing the angle of the rigid Headgear on your face by shortening one Headgear strap and lengthening the other.

CLEANING YOUR MASK AT HOME

To ensure continuing safety and comfort from your Opus™ Nasal Pillows Mask:

- BEFORE EACH USE:**
 - Inspect the mask for deterioration. Do not use if damaged.
 - Inspect the Air Vents (V). Do not use if blocked.
 - AFTER EACH USE:**
 - Remove the Silicone Pillows (C) from the Mask Base (A) and disconnect the Mask Base from the Headgear (D). Disconnect the Male Swivel (I) from the end of the Flexitube (B).
 - Wash the Silicone Pillows (C), Mask Base (A) with Flexitube (B) and Male Swivel (I) in pure soap dissolved in lukewarm water. Do not soak for longer than 10 minutes. Rinse with fresh water.
 - Leave all parts to dry out of direct sunlight before reassembling.
 - EVERY 7 DAYS:**
 - Hand-wash the Headgear (D) in pure soap dissolved in lukewarm water. Rinse in fresh water and allow to dry out of direct sunlight.
- CAUTION**
- Do not soak for longer than 10 minutes.
 - Do not wash the mask in the dishwasher.
 - Do not clean the mask with products containing alcohol, anti-bacterial agents, antiseptic, bleach, chlorine or moisturizer.
 - Do not store the mask in direct sunlight.
- The above actions may deteriorate or damage the mask and shorten its life. If your Opus™ Nasal Pillows Mask weakens or cracks, discontinue use and seek replacement immediately.

ASSEMBLING YOUR MASK

Your Opus™ Nasal Pillows Mask is fully assembled and ready to use. However, when you need to clean your mask, follow these instructions for reassembly:

- Push the Male Swivel (I) onto the end of the Flexitube (B).
- Insert the Silicone Pillows (C) into the channel on the Mask Base (A) as shown in diagram (W). Ensure the triangular arrow on the Silicone Pillows aligns with the triangular recess in the Mask Base.
- Slide the Headgear (D) into the curved hook on the Mask Base (A) as shown in diagram (X). Ensure the Headgear is firmly in place.
- Use the Tube Anchoring Strap (L) to fix the tube to the Headgear in the desired location. Alternatively, you can leave the tube unanchored.

Top position: Feed the Tube Anchoring Strap (L) around the middle bar of the Tri-glide (T) and position the Flexitube (B) as shown in diagram (Y). Feed the pointed end of the strap through the small hole in the other end of the strap and pull tight around the Flexitube.

Side position: Feed the Tube Anchoring Strap between the grey rigid component of the Headgear and the black Breath-O-Prene® material at the desired side anchor point as shown in diagram (Z). Position the Flexitube (B). Feed the pointed end of the strap through the small hole in the other end of the strap and pull tight around the tube.

OXYGEN/PRESSURE PORT CONNECTOR

If pressure readings and/or additional oxygen are required, an Oxygen/Pressure Port Connector is available (Code 900HC452).

- Connect the large end of the Connector (T) to the Swivel (G) on the mask and the small end of the Connector to the main breathing tube (R).
- Open the Port Cap(s) (P) and place the Pressure and/or Oxygen Tube (S) firmly over the Port(s) (Q).

(The two Ports are identical; either Port may be used for oxygen or pressure measurement. Only remove the Port Cap from Ports that are being used.)

- Note:** At a fixed flow rate of supplemental oxygen flow, the inhaled oxygen concentration will vary, depending on the pressure settings, patient's breathing pattern, mask selection, and the leak rate.
- Warning:** If oxygen is used with CPAP/Bi-Level machine, the oxygen flow must be turned off when the CPAP/Bi-Level machine is not operating.
- Explanation of Warning:** When the CPAP/Bi-Level device is not in operation, and the oxygen flow is left on, oxygen delivered into the ventilator tubing may accumulate within the CPAP/Bi-Level machine enclosure. Oxygen accumulated in the CPAP/Bi-Level machine enclosure will create a risk of fire.
- Warning:** Do not smoke while using or in the vicinity of someone using oxygen with this mask.
- Warning:** Do not use Polyvinyl Chloride (PVC) Pressure and/or Oxygen tubing. Use silicone tubing instead.
- Explanation of Warning:** Contact with PVC can cause premature cracking of ports.

WARNINGS

- This mask should be used only with CPAP or Bi-Level equipment recommended by your physician or respiratory therapist. A mask should not be used unless the CPAP or Bi-Level machine is turned on and operating properly. The vent holes associated with the mask should never be blocked.
- Explanation of Warning:** CPAP and Bi-Level machines are intended to be used with special masks which have exhalation ports to allow continuous flow of air out of the mask. When the CPAP/Bi-Level machine is turned on and functioning properly, new air from the CPAP/Bi-Level machine flushes the exhaled air out through the mask exhalation ports. However, when the CPAP/Bi-Level machine is not operating, enough fresh air will not be provided through the mask, and exhaled air may be re-breathed. Re-breathing of exhaled air for longer than several minutes can in some circumstances lead to suffocation.
- At low CPAP pressures the flow through the exhalation ports may be inadequate to clear all exhaled gas from the tubing. Some re-breathing may occur.
- Do not use the mask if vomiting or feeling nauseous.
- Discontinue use if discomfort or irritation occurs, and consult your Healthcare Provider.
- Discontinue using the mask if there is an allergic reaction to any part of the mask. Consult your physician if this occurs.

CAUTION

- Use the mask for its intended use only as directed in this booklet.
- US Federal Law restricts this device to sale by, or on order of, a physician.

The Opus™ Nasal Pillows Mask and accessories are Latex Free.

PATENTS PENDING

TECHNICAL SPECIFICATIONS

EXHAUST FLOW	
The Opus™ Nasal Pillows Mask has an air exhaust system that vents the expelled CO ₂ through a group of fine holes in the front of the Elbow.	
It is important that these holes are not blocked by any object. This controlled leak ensures exhaled CO ₂ is expelled from the mask.	
PRESSURE (cmH ₂ O)	Flow (L/min)
3	17
5	22
7	26
9	30
11	33
13	36
15	39
17	42
19	45
21	48
23	50
25	52
RESISTANCE TO FLOW	
MEDIUM	
Pressure drop through mask @ 50 l/min 1.1 cmH ₂ O	
Pressure drop through mask @ 100 l/min 3.9 cmH ₂ O	
MASK DEAD SPACE	
T5cc	
OPERATING CONDITIONS	
Operating pressure is 3-25 cm H ₂ O	
Operating temperature range is 5-40°C (40-104°F)	

OPUS™ NASAL PILLOWS MASK MULTI-PATIENT CLEANING INSTRUCTIONS

Your Opus™ Nasal Pillows Mask is provided assembled and ready for patient use. To reuse between patients please follow the guidelines below.


CLEANING / DE-SOILING

- Before disinfecting, disassemble all components from the Mask Base (the Headgear, Silicone Pillows and Male Swivel). Clean all surfaces of the mask and components thoroughly with a mild detergent solution and a soft bristled brush. Ensure all visible soil is removed. Rinse thoroughly under running potable water to remove detergent solution.

FAILURE TO PROPERLY CLEAN THIS DEVICE MAY RESULT IN INADEQUATE DISINFECTION.

HIGH LEVEL DISINFECTION PARAMETERS

MASK BASE AND FLEXITUBE	HERMAL DISINFECTION
	Soak parts in hot water bath for: 30 minutes at 70°C (158°F) or 30 minutes at 75°C (167°F)
	CIDEX® OPA Follow manufacturer's instructions for high level disinfection. Immerse for 30 minutes at 20°C (68°F). Rinse thoroughly in potable water.
	STERRAD® 100S Follow manufacturer's instructions for Sterrad® 100S Sterilization system.

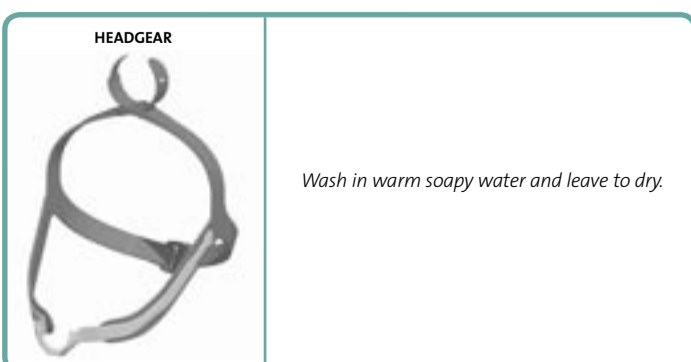THE OPUS™ NASAL PILLOWS MASK HAS BEEN VALIDATED TO WITHSTAND 20 CYCLES OF THE ABOVE PARAMETERS. DO NOT USE IF DAMAGED.

Disclaimer: It is the user's responsibility to qualify any deviations from the recommended methods of reprocessing.

CONTACT YOUR LOCAL HOME CARE PROVIDER FOR ADDITIONAL INFORMATION.

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