



ResMed Mirage Quattro Fitting Template

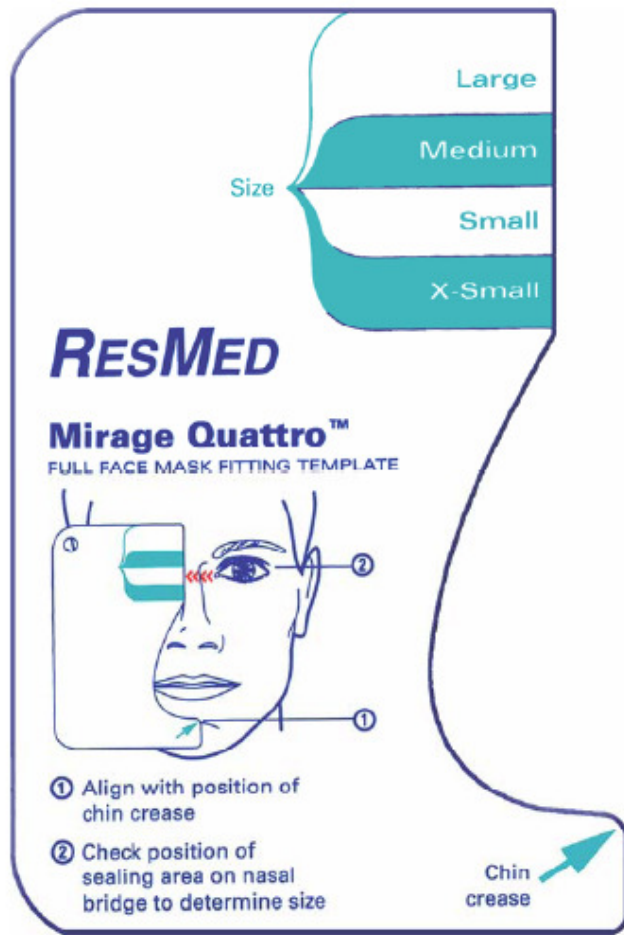

1. Print this page
2. Compare Inch marker to a ruler ensure correct scale
3. Cut around the perimeter of the fitting template
4. Follow instructions on fitting template to determine your correct size. See notes below for more detailed instruction.

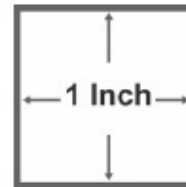

- ① Align with position of chin crease
- ② Check position of sealing area on nasal bridge to determine size

Helpful hints for determining your size:

1. If patient's weight tends to fluctuate, select a larger size
2. If patient is going to have bariatric surgery, select a smaller size
3. If patient has facial hair, select a larger size
4. If patient is a restless sleeper during sleep, select a larger size

To determine the correct cushion size:

1. First align the inside bottom corner (arrow) with the crease in your chin.
2. The correct cushion size will be aligned with the inside corner of your eye.

