

Brand	Style	Mask Size	Liner Size	Fit Rating	Notes
ResMed	AirFit F10	XS	Small	Excellent	
ResMed	AirFit F10	S	Small	Excellent	
ResMed	AirFit F10	M	Medium	Great	
ResMed	AirFit F10	L	Large	Great	
ResMed	AirFit F20	S	Small	Excellent	
ResMed	AirFit F20	M	Medium	Excellent	
ResMed	AirFit F20	L	Large	Excellent	
ResMed	AirTouch F20	S	Small	Excellent	
ResMed	AirTouch F20	M	Medium	Excellent	
ResMed	AirTouch F20	L	Large	Excellent	

ResMed	Mirage Quattro	XS	Small	Excellent	
ResMed	Mirage Quattro	S	Small	Excellent	
ResMed	Mirage Quattro	M	Medium or Large	Excellent	
ResMed	Mirage Quattro	L	Large	Excellent	
ResMed	Quattro FX	S	Small	Excellent	Attach upper Quick Stick tabs under head gear wings.
ResMed	Quattro FX	M	Small or Medium	Great	Medium size liner fits a little loose. Small size liner fits nicely but has a slightly smaller mouth hole. Attach upper Quick Stick tabs under head gear wings.
ResMed	Quattro FX	L	Medium or Large	Great	Large size liner fits a little loose. Medium size liner fits nicely. Attach upper Quick Stick tabs under head gear wings.
ResMed	Quattro Air	XS	Small	Excellent	
ResMed	Quattro Air	S	Small	Excellent	
ResMed	Quattro Air	M	Medium	Excellent	
ResMed	Quattro Air	L	Large	Excellent	
ResMed	Ultra Mirage	S	Small	Excellent	
ResMed	Ultra Mirage	M	Medium	Excellent	

ResMed	Ultra Mirage	L	Large	Excellent	
Respironics	Amara View	S	One-Size-Fits-All	Excellent	
Respironics	Amara View	M	One-Size-Fits-All	Excellent	
Respironics	Amara View	L	One-Size-Fits-All	Excellent	
Respironics	Amara Full	P	Small	Excellent	
Respironics	Amara Full	S	Small	Excellent	
Respironics	Amara Full	M	Medium	Excellent	
Respironics	Amara Full	L	Large	Excellent	
Respironics	Comfort Gel Full	S	Small	Excellent	
Respironics	Comfort Gel Full	M	Medium	Excellent	
Respironics	Comfort Gel Full	L	Large	Excellent	
Respironics	DreamWear	S	One Size Fits-All		Quick Stick tabs may only partially engage with the hard plastic.
Respironics	DreamWear	M	One-Size-Fits-All	Excellent	Quick Stick tabs may only partially engage with the hard plastic.

Respironics	DreamWear	MW	One-Size-Fits-All	Great	Quick Stick tabs may only partially engage with the hard plastic.
Respironics	Amara View	L	One-Size-Fits-All	Great	Quick Stick tabs may only partially engage with the hard plastic.
Respironics	FitLife	S	Small	Excellent	The FitLife liner was developed specifically for the Respironics FitLife mask.
Respironics	FitLife	L/XL	L/XL	Excellent	The FitLife liner was developed specifically for the Respironics FitLife mask.
Respironics	Full Life	S	Small	Poor	Quick Stick tabs do not reach hard plastic.
Respironics	Full Life	M	Medium	Poor	Quick Stick tabs do not reach hard plastic.
Respironics	Full Life	L	Large	Poor	Quick Stick tabs do not reach hard plastic.
Fisher & Paykel	Forma	S	Large	Great	The lower Quick Stick tabs of the Large liner will be able to reach the hard plastic on the bottom side of the mask (preferred attaching location). Even a small amount of adhesive contact to the hard plastic is adequate to hold the liner in place.
Fisher & Paykel	Forma	M	Large	Great	The lower Quick Stick tabs of the Large liner will be able to reach the hard plastic on the bottom side of the mask (preferred attaching location). Even a small amount of adhesive contact to the hard plastic is adequate to hold the liner in place.
Fisher & Paykel	Forma	L	Large	Great	The lower Quick Stick tabs of the Large liner will be able to reach the hard plastic on the bottom side of the mask (preferred attaching location). Even a small amount of adhesive contact to the hard plastic is adequate to hold the liner in place.
Fisher & Paykel	Flexifit 432	S	Large	Great	The lower Quick Stick tabs of the Large liner will be able to reach the hard plastic on the bottom side of the mask (preferred attaching location). Even a small amount of adhesive contact to the hard plastic is adequate to hold the liner in place.
Fisher & Paykel	Flexifit 432	M	Large	Great	The lower Quick Stick tabs of the Large liner will be able to reach the hard plastic on the bottom side of the mask (preferred attaching location). Even a small amount of adhesive contact to the hard plastic is adequate to hold the liner in place.
Fisher & Paykel	Flexifit 432	L	Large	Great	The lower Quick Stick tabs of the Large liner will be able to reach the hard plastic on the bottom side of the mask (preferred attaching location). Even a small amount of adhesive contact to the hard plastic is adequate to hold the liner in place.

Fisher & Paykel	Simplus	S	Small	Excellent	
Fisher & Paykel	Simplus	M	Medium	Excellent	
Fisher & Paykel	Simplus	L	Large	Excellent	
Drive	Wizard 220	M	Large	Excellent	
Drive	Wizard 220	L	Large	Great	The lower Quick Stick tabs of the Large liner will be able to reach the hard plastic (preferred attaching location) on the bottom side of the mask. Even a small amount of adhesive contact to the hard plastic is adequate to hold the liner in place.
Hoffrichter	Cirri	S	S	Excellent	
Hoffrichter	Cirri	M	M	Excellent	
Hoffrichter	Cirri	L	L	Excellent	
Hoffrichter	Comfo	S	S	Excellent	The Quick Stick tabs need to engage with hard plastic on the mask for best attachment.
Hoffrichter	Comfo	M	M	Great	The Quick Stick tabs need to engage with hard plastic on the mask for best attachment.
Hoffrichter	Comfo	L	L	Great	The Quick Stick tabs need to engage with hard plastic on the mask for best attachment.
Sleepnet	Ascend	S	Small	Excellent	Align the top edge of the liner with the top of the gel cushion.
Sleepnet	Ascend	M	Medium	Excellent	Align the top edge of the liner with the top of the gel cushion.

Sleepnet	Ascend	L	Large	Excellent	Align the top edge of the liner with the top of the gel cushion.
Sleepnet	Mojo	S	Small	Excellent	Align the top edge of the liner with the top of the gel cushion.
Sleepnet	Mojo	M	Medium	Excellent	Align the top edge of the liner with the top of the gel cushion.
Sleepnet	Mojo	L	Large	Excellent	Align the top edge of the liner with the top of the gel cushion.